

# DETECTO PHR Portable Height Rod Assembly and Operation

## Assembly

① Open insert. Slide Headpiece, and lift to remove it. Locate and remove Stabilizers from carton.



② Lift insert to side to fully expose Base and using both hands remove Base from carton.



③ Remove Column Pieces from carton.



④ Verify carton contents: Base, Headpiece, (4) Column Pieces and (2) stabilizers.



⑤ Place PHR base on floor.




⑥ Install (1) Stabilizer on Column Piece "A" (with large arrow) and insert into Base as shown below.



⑦ Insert Column Piece marked "A-B" into Column Piece "A" and press down to lock in place.



⑧ Insert Column Piece marked "B-C" into Column Piece "A-B" and press down to lock in place.

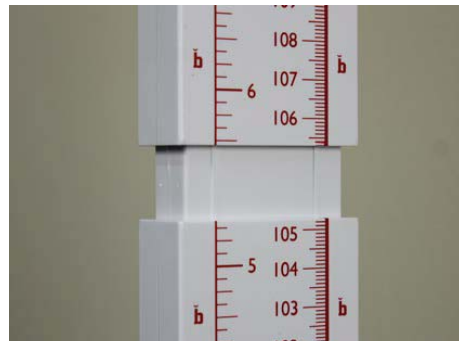


## Assembly, Cont.

⑨ Install (1) Stabilizer on Column Piece “C” and insert into Headpiece as shown below.



⑩ Insert Column Piece marked “C” into Column Piece “B-C” and press down to lock in place.

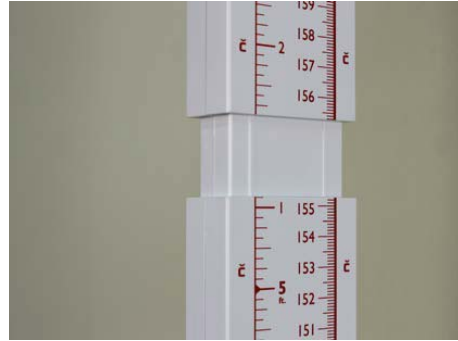

Assembly is complete. The PHR Portable Height Rod is ready to use.

## Operation



1. Position upper and lowers Stabilizers on the Column and place PHR against the wall.
2. Raise Headpiece well above estimated height of patient.
3. Assist patient onto PHR Base (or scale). Note that patient should stand straight up with their back should slightly touching ruler.
4. Pull headpiece down to patient's head and read height. Height is read at red arrows on headpiece.



Headpiece installed for use without scale.



Headpiece installed for use with scale.

# DETECTO®

DETECTO  
102 E. Daugherty, Webb City, MO 64870 USA  
PH: 417-673-4631 • FX: 417-673-2153  
www.detecto.com

0044-0392-0M Rev A 09/15