



Installation Instructions for Stretcher Scale Guide Rails

(8500-GUIDERAILS for Models 8500/8550 Stretcher Scales)

UNPACKING

Before starting to assemble, please unpack and remove all wrappings and fillers. Carton contains:

Item Description	Part Number	Qty.
8500/8550 Guide Rail, Wheel Side	0065-0824-08	2
8500/8550 Guide Rail, Handle Side	0065-0825-08	2
Screw Truss-Head Thread Cutting, 8-32 x .375	6021-1257	10

TOOLS REQUIRED

- No. 2 Philips screwdriver (power screwdriver recommended)

ASSEMBLY

Refer to Figures No. 1, 2, and 3, then follow the instructions for attaching the Guide Rails.

- Place the platform on the floor in the *storage* position.
- Begin by holding a Wheel Side Guide Rail (with caution label toward you) against the side of the platform (this is the longer of the two sets of guide rails). Install (2) thread cutting screws through the holes in the guide rail, and into the holes of the platform. Securely tighten the screws. Refer to Figure No. 1.
- Repeat step 2 for the other Wheel Side Guide Rail.
- Next, hold a Handle Side Guide Rail (with caution label toward you) against the side of the platform (this is the shorter of the two sets of guide rails). Install (3) thread cutting screws through the holes in the guide rail, and into the holes of the platform. Securely tighten the screws. Refer to Figure No. 2.
- Repeat step 4 for the other Handle Side Guide Rail.
- Finished assembly should look like Figure No. 3.



Figure No. 1



Figure No. 2



Figure No. 3

DETECTO
102 E. Daugherty, Webb City, MO 64870 USA
Ph: 417-673-4631 or 1-800-641-2008
Fax: 417-673-2153
www.Detecto.com
Technical Support: 1-866-254-8261
E-mail: tech@cardet.com