# DETECTO®

## **SlimTalk/SlimTalkXL**

**Talking Home Health Scale  
Operation Manual**

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## Proper Disposal

When this device reaches the end of its useful life, it must be properly disposed of. It must not be disposed of as unsorted municipal waste. Within the European Union, this device should be returned to the distributor from where it was purchased for proper disposal. This is in accordance with EU Directive 2002/96/EC. Within North America, the device should be disposed of in accordance with the local laws regarding the disposal of waste electrical and electronic equipment.

It is everyone's responsibility to help maintain the environment and to reduce the effects of hazardous substances contained in electrical and electronic equipment on human health. Please do your part by making certain that the device is properly disposed of. The symbol shown to the right indicates that this device must not be disposed of in unsorted municipal waste programs.

## FCC Compliance Statement

This equipment generates uses, can radiate radio frequency, and if not installed and used in accordance with the instruction manual, may cause interference to radio communications. It has been designed within the limits for a Class A computing device pursuant to Subpart J of Part 15 of FCC rules to provide reasonable protection against such interference when operated in a commercial environment. Operation of this equipment in a residential area may cause interference in which case the user will be responsible for taking whatever measures necessary to correct the interference.

You may find the booklet "How to Identify and Resolve Radio-TV Interference Problems" prepared by the Federal Communications Commission helpful. It is available from the U.S. Government Printing Office, Washington, D.C. 20402. Request stock No. 001-000-00315-4.

## Copyright

All rights reserved. Reproduction or use, without express written permission, of editorial or pictorial content, in any manner, is prohibited. No patent liability is assumed with respect to the use of the information contained herein.

## Disclaimer

While every precaution has been taken in the preparation of this manual, the Seller assumes no responsibility for errors or omissions. Neither is any liability assumed for damages resulting from the use of the information contained herein. All instructions and diagrams have been checked for accuracy and ease of application; however, success and safety in working with tools depend to a great extent upon individual accuracy, skill, and caution. For this reason, the Seller is not able to guarantee the result of any procedure contained herein. Nor can they assume responsibility for any damage to property or injury to persons occasioned by the procedures. Persons engaging in the procedures do so entirely at their own risk.

## Features

Capacity SlimTALK SlimTALK XL	400 lb x 0.1 lb / 180 kg x 0.1kg 550 lb x 0.1 lb / 250 kg x 0.1kg
Weight Indication Display Voice	Four-digit, seven-segment LCD with 1.5 in / 38 mm high digits English / Spanish / Off
Indicators	lb, kg, Low Battery
Functions	Auto-ON (minimum weight of 11 lb / 5 kg is required) Auto-OFF (10 seconds)
Power	Included 3 “AAA” Batteries (4.5 VDC)

## Unpacking

Before using your scale, make certain the scale has been received in good condition. When removing the scale from its packing, inspect it for signs of damage, such as exterior dents and scratches. Keep the carton and packing material for return shipment if it should become necessary. It is the responsibility of the purchaser to file all claims for any damages or loss incurred during transit.

SlimTALK

## Power-Up Initialization

1. Turn the scale over and locate the removable panel on the bottom of the scale.
2. Remove the panel to gain access to the battery holder.
3. Install the included three (3) “AAA” size batteries.
4. Place the scale on a flat-level surface, such as a desk, table, or bench.
5. Press on the scale to turn it on.

**NOTE:** A minimum weight of 11 lb / 5 kg is required to turn the scale on.

6. The scale will display **0.0** for approximately 10 seconds, then turn off.
7. The scale has completed initialization and is ready for normal operation.

SlimTALK XL

## Operation

1. Step on the scale. It will turn on automatically.
2. The scale will measure weight. Once stable, the display will blink several times and then lock on weight. If voice mode is on, the scale will announce weight (in the selected language).
3. Step off scale. Weight will continue to be displayed for approximately 10 seconds.
4. The scale will then automatically turn off. If voice mode is on, the scale will say “Goodbye” (in the selected language) when turning off.

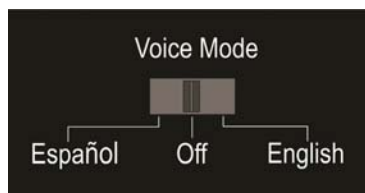
## Weighing Unit Switch

The weighing unit switch (lb/kg switch) is a push-button switch located on the bottom of the scale. With the scale off, press the switch to toggle between lb and kg.

## Voice Language Switch

Move the switch to select language or turn the voice off.

## Low Battery

When the battery voltage has dropped too low for accurate weighing, the  $L o$  message will be displayed and the scale will automatically turn off. Remove the old batteries and replace them with new batteries before continuing operation.

## Error and Status Messages

**NOTE:** The error messages are displayed for approximately 2 seconds, and then the scale will turn off.

Display	Meaning
$E r r$	Over Capacity
$L o$	Battery Low

## Troubleshooting Guide

The following guide will assist you in determining errors and problems you may encounter with your DETECTO® Scale SlimTalk or SlimTalkXL talking digital bathroom scale.

SYMPTOM	POSSIBLE CAUSE	POSSIBLE SOLUTION
Err is displayed briefly and the scale turns off.	Weight on scale exceeds scale capacity.	Ensure the scale is empty and not touching anything before weighing a patient. If the scale continues to show Err, contact your scale service representative.
Lo is displayed briefly and the scale turns off.	One or more batteries are too low for accurate weighing and should be replaced.	Remove the battery panel on the bottom of the scale. Replace all (3) batteries, and then re-install the battery panel.
The scale will not turn on after replacing the batteries.	One or more batteries may be installed incorrectly.	Remove the battery panel on the bottom of the scale. Verify that all (3) batteries are installed correctly in the battery holder, and then re-install the battery panel.
The scale is displaying the wrong weighing units.	Push-button weighing units switch accidentally pushed while handling the scale or when changing batteries.	Ensure the scale is off. Press the weighing units switch on the bottom of the scale to toggle to correct weighing units.
Scale is speaking weight in the wrong language or not at all.	The Voice Language switch accidentally moved while handling the scale or when changing batteries.	On the bottom of the scale, move the Voice Mode switch to the desired language.
The weight displayed is wrong or display erratic.	<p>Is the scale used on a carpeted or non-carpeted floor?</p> <p>Carpet feet should be installed when the scale is used on a carpeted floor.</p> <p>On a non-carpeted floor, the carpet feet should not be installed.</p>	<p>If scale is used on carpeted floor, check if carpet feet are installed. If not, install carpet feet and weigh the patient again.</p> <p>If the scale continues to show wrong or erratic weight, move the scale to a non-carpeted floor and weigh the patient again.</p> <p>If on a non-carpeted floor, the scale continues to show wrong or erratic weight, contact your scale service representative.</p>

## Installing the Carpet Feet

If you are using your scale on a carpet, we recommend that you use the included carpet feet. To install the carpet feet, perform the following:

1. Remove the four (4) carpet feet from the packaging.
2. Align the prongs of one carpet foot with the holes in one of the scale's existing feet.
3. Insert the carpet foot prongs into the holes, making sure the prongs align straight up and down with the holes of the scale foot.
4. Being careful not to twist the carpet foot, with even pressure, gently push straight down on the carpet foot to seat it into the scale foot.
5. Repeat steps 2 through 4 for the other three (3) scale feet.

## STATEMENT OF LIMITED WARRANTY

DETECTO warrants its equipment to be free from defects in material and workmanship as follows: DETECTO warrants to the original purchaser only that it will repair or replace any part of equipment which is defective in material or workmanship for a period of two **(2) years from date of shipment**. DETECTO shall be the sole judge of what constitutes a defect.

During the **first ninety (90) days** DETECTO may choose to replace the product at no charge to the buyer upon inspection of the returned item.

**After the first ninety (90) days**, upon inspection of the returned item, DETECTO will repair or replace it with a remanufactured product. The customer is responsible for paying for the freight both ways.

This warranty does not apply to peripheral equipment not manufactured by DETECTO; this equipment will be covered by certain manufacturer's warranty only.

This warranty does not include replacement of expendable or consumable parts. This does not apply to any item which has deteriorated or damaged due to wear, accident, misuse, abuse, improper line voltage, overloading, theft, lightning, fire, water or acts of God, or due to extended storage or exposure while in purchaser's possession. This warranty does not apply to maintenance service. Purchased parts will have a ninety (90) day repair or replacement warranty only.

DETECTO may require the suspect product to be returned to the factory; item(s) must be properly packed and shipping charges prepaid. A return authorization number must be obtained for all returns and marked on the outside of all returned packages. DETECTO accepts no responsibility for loss or damage in transit.

# STATEMENT OF LIMITED WARRANTY

## Conditions Which Void Limited Warranty

This warranty shall not apply to equipment which:

- A.) Has been tampered with, defaced, mishandled, or has had repairs and modifications not authorized by DETECTO.
- B.) Has had serial number altered, defaced, or removed.
- C.) Has not been grounded according to DETECTO's recommended procedure.

## Freight Carrier Damage

Claims for equipment damaged in transit must be referred to the freight carrier in accordance with freight carrier regulations.

This warranty sets forth the extent of our liability for breach of any warranty or deficiency in connection with the sale or use of the product. DETECTO will not be liable for consequential damages of any nature, including but not limited to, loss of profit, delays, or expenses, whether based on tort or contract. DETECTO reserves the right to incorporate improvements in material and design without notice and is not obligated to incorporate improvements in equipment previously manufactured.

The foregoing is in lieu of all other warranties, express or implied including any warranty that extends beyond the description of the product including any warranty of merchantability or fitness for a particular purpose. This warranty covers only those DETECTO products installed in the forty-eight (48) contiguous continental United States.

Ph. (800) 641-2008  
E-mail: [detecto@cardet.com](mailto:detecto@cardet.com)  
102 E. Daugherty  
Webb City, MO 64870

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