



## WPS12DT Wet Diaper Scale

### Attachment of Diaper Tray to Scale Platter

1. Remove platter (A) from scale and clean with alcohol or similar cleaner.
2. On bottom of diaper tray (B) remove protective backing from adhesive side of Velcro.
3. Press diaper tray (B) onto scale platter (A) so that adhesive attaches Velcro strips to scale platter.
4. Place diaper tray scale platter assembly back onto scale.
4. Diaper tray may be removed from scale platter by gently pulling up on the tray separating Velcro strips.

