

DETECTO®



HS25KGP **Suspended Sling Seat** **Baby Scale** **Owner's Manual**

UNPACKING


1. Open box.
2. Remove top Styrofoam insert from box.
3. Next, remove S-hook from insert and set aside.
4. Grasp scale and remove from box.
5. Remove Pants (baby sling seat) from box.
6. Remove bottom Styrofoam insert from box.
7. Turn insert over to locate second S-hook.
8. Remove S-hook from insert and set aside.
9. Remove scale from plastic bag.
10. Remove cardboard dowel between scale case and lower loop.
11. Scale is now ready to use.



Figure No. 1

OPERATION

1. Referring to Figure No. 1, install S-hooks on scale.
2. Unfold Pants (Baby Sling Seat).
3. Hang Pants on lower S-hook. See Figure No. 1.
4. Referring to Figure No. 2, turn Zero Adjust Knob until dial pointer is reading "0".
5. Place baby in Pants.
6. The dial pointer will indicate the baby's weight.
7. Remove baby from Pants and re-zero scale if necessary.



NEVER leave a baby unattended in the **Baby Sling Seat**. Failure to maintain control of the baby at all times while they are in the **Baby Sling Seat** can result in serious injury to baby.



Figure No. 2

CARE AND CLEANING

- **DO NOT** spray; water directly on the scale.
- **DO** clean scale using a damp, soft cloth and mild detergent.
NOTE! Some types of paper towels may scratch the dial lens.
- **DO NOT** use acetone, alcohol, or other volatile solvents for cleaning.

PARTS LIST

ITEM	QTY	PART NUMBER	DESCRIPTION
1	4	6600-1870	Lens Screw
2	1	6600-1871	Lens
3	2	6600-1872	S-Hook
4	1	HSPANT	Pants (Baby Sling Seat)

STATEMENT OF LIMITED WARRANTY

DETECTO warrants its equipment to be free from defects in material and workmanship as follows: DETECTO warrants to the original purchaser only that it will repair or replace any part of equipment which is defective in material or workmanship for a period of **two (2) years from date of shipment**. Detecto shall be the sole judge of what constitutes a defect.

During the **first ninety (90) days** DETECTO may choose to replace the product at no charge to the buyer upon inspection of the returned item.

After the first ninety (90) days, upon inspection of the returned item, DETECTO will repair or replace it with a remanufactured product. The customer is responsible for paying for the freight both ways.

This warranty does not apply to peripheral equipment not manufactured by DETECTO; this equipment will be covered by certain manufacturer's warranty only.

This warranty does not include replacement of expendable or consumable parts. This does not apply to any item which has deteriorated or damaged due to wear, accident, misuse, abuse, improper line voltage, overloading, theft, lightning, fire, water or acts of God, or due to extended storage or exposure while in purchaser's possession. This warranty does not apply to maintenance service. Purchased parts will have a ninety (90) day repair or replacement warranty only.

DETECTO may require the product to be returned to the factory; item(s) must be properly packed and shipping charges prepaid. A return authorization number must be obtained for all returns and marked on the outside of all returned packages. DETECTO accepts no responsibility for items lost or damaged in transit.

Conditions Which Void Limited Warranty

This warranty shall not apply to equipment which:

- A.) Has been tampered with, defaced, mishandled or has had repairs and modifications not authorized by DETECTO.
- B.) Has had serial number altered, defaced, or removed.
- C.) Has not been properly grounded according to Detecto's recommended procedure.

Freight Carrier Damage

Claims for equipment damaged in transit must be referred to the freight carrier in accordance with freight carrier regulations.

This warranty sets forth the extent of our liability for breach of any warranty or deficiency in connection with the sale or use of the product. DETECTO will not be liable for consequential damages of any nature, including but not limited to, loss of profit, delays or expenses, whether based on tort or contract. Detecto reserves the right to incorporate improvements in material and design without notice and is not obligated to incorporate improvements in equipment previously manufactured.

The foregoing is in lieu of all other warranties, express or implied including any warranty that extends beyond the description of the product including any warranty of merchantability or fitness for a particular purpose. This warranty covers only those DETECTO products installed in the forty-eight (48) contiguous continental United States.



Ph. (800) 641-2008
E-mail: detecto@cardet.com
102 E. Daugherty
Webb City, MO 64870

04/22/2024
Printed in USA
D268-WARRANTY-DET-B



DETECTO

102 E. Daugherty, Webb City, MO 64870 USA

Ph: 417-673-4631 or 1-800-641-2008

Fax: 417-673-2153

www.Detecto.com

Technical Support: 1-866-254-8261

E-mail: tech@cardet.com

Printed in USA

8535-M214-O1 Rev B 03/15