



# Model DR660 Calibration Instructions

Before beginning calibration, the following equipment is required:

Calibrated test weights    200 kg or 500 lbs

## To begin calibration:

1. Make certain scale platform is empty.
2. With the scale off, press and hold the **ON/ZERO/OFF** key.
3. Press **kg/lb** key three times and then release both keys.
4. Scale display will show *2000 ~ 9000* count.
5. Now press **kg/lb** key to select kg or lb to calibrate the scale.
6. Press **TARE** key.
7. When scale display shows *0.0*, place 200kg (or 500lb) weight on the platform.
8. Scale display will show *8000 ~ 14000* count.
9. Press **TARE** key.
10. Scale display will show *CAL* then *200* (or *500*).
11. The calibration is finished.
12. Remove test weights from scale platform.
13. Turn scale off and then back on.
14. Scale is now ready for operation.

DETECTO

102 E. Daugherty, Webb City, MO 64870 USA

Ph: 417-673-4631 or 1-800-641-2008 • Fax: 417-673-2153

[www.Detecto.com](http://www.Detecto.com)

Technical Support: 1-866-254-8261 • E-mail: [tech@cardet.com](mailto:tech@cardet.com)