

PZ30W/PZ60W

Calibration Procedure

1. Power on display and base for normal weighing.
2. Press and hold **UNITS** key until $\overline{CAL}$ is shown on display.
3. Display will then alternate between $\overline{Load}$ and a calibration weight.

PZ30W

$\overline{Load}$ and 20.00 lb or $\overline{Load}$ and 10.00 kg

PZ60W

$\overline{Load}$ and 50.00 lb or $\overline{Load}$ and 20.00 kg

To switch between LB and KG press **UNITS** key once.

4. Load scale to displayed weight.
5. Display will change to show $\overline{CAL}$.
6. Once calibration is complete, scale will return to normal weigh mode.
7. Remove weight from scale.
8. Scale is now calibrated and ready for normal use.