



SlimPRO Calibration Instructions

1. Press and hold **UNIT** button (on the bottom of the scale).
2. Install the batteries. All display segments will turn on.
3. Press **UNIT** button 4 times quickly and hold **UNIT** button on the fourth press until the scale shows the zero count.
4. Press **UNIT** button, and place the scale on the table or bench in 4 seconds.
5. The scale will show "0.0 kg" and then "65.0 kg".
6. Put 65kg of calibrated test weight on the scale.
7. The scale will show "135.0 kg".
8. Put 135kg of calibrated test weight on the scale.
9. The scale will show "PASS", and then power off automatically.