SlimPRO

Low Profile Health Scale Operation Manual

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Technical Support: Ph: 866-254-8261 • tech@cardet.com

Features

- 4 x 7-segment LCD display
- Capacity: 440 lbs (200 kg)
- Graduations: 0 to 440 lbs x 0.2 lb (0 to 200 kg x 100 g)
- Auto-On function
- Auto-OFF function
- Low battery indicator
- RS232 Output Port (Requires optionally purchased interface cable)

Unpacking

Before using your scale, make certain the scale has been received in good condition. When removing the scale from its packing, inspect it for signs of damage, such as exterior dents and scratches. Keep the carton and packing material for return shipment if it should become necessary. It is the responsibility of the purchaser to file all claims for any damages or loss incurred during transit.

Power-Up Initialization

1. Turn the scale over and locate the removable panel on the bottom of the scale.
2. Remove the panel to gain access to the battery holder.
3. Install the included four (4) “AAA” size alkaline batteries.
4. Place the scale on a flat level surface, such as a desk, table or bench.
5. Lightly press on the scale to turn it on.
6. The scale display will show **0000**, change to a moving **0** and then turn off.
7. The scale has completed initialization and is ready for normal operation.

Operation

1. Step on the scale.
2. The scale shows “0.0” during wake up and then shows the users weight.

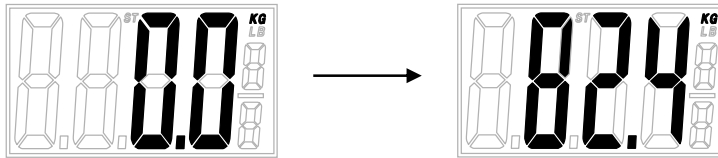
Fig. 1: Measured Weight Screens

3. The weight is locked once a stable weight is obtained.
4. If the user steps off the scale before a stable weight is obtained, the scale will display 0.0lb/kg for 5 seconds, show a moving zero (see Fig. 2: below) and then the scale will automatically turn OFF.
5. When the user steps off the scale after a stable weight is obtained, the users weight will be displayed for 5 seconds, then a moving zero will be shown (see Fig. 2: below) and then the scale will automatically turn OFF.
6. If the user is still on the scale over 20 seconds, the scale will automatically turn OFF without showing the moving zero.

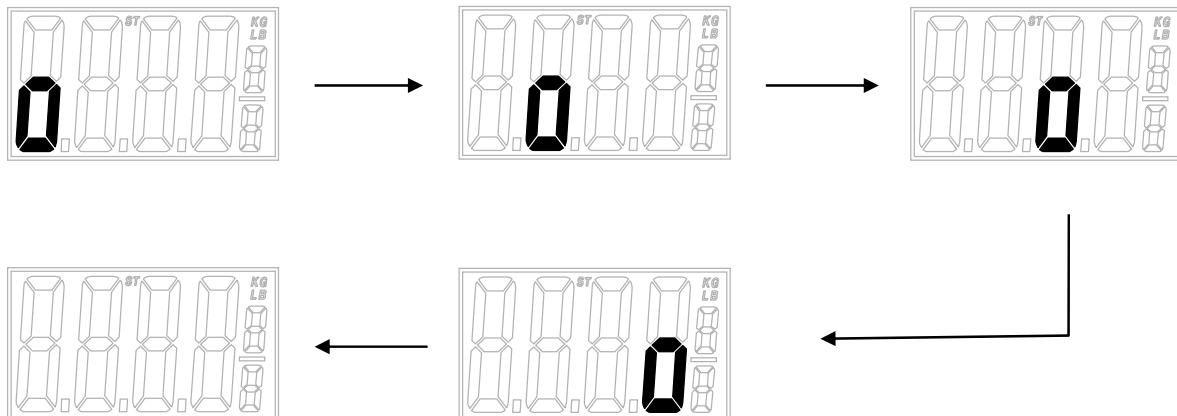
Fig. 2: Moving Zero

Weighing Unit Switch


The weighing unit switch (kg /lb switch) is a push-button switch located on the bottom of the scale. Press the switch to toggle between kg and lb. Note that the default setting unit is kg.

1. With the scale off, press the push-button.
2. Zero weight and the current weighing unit setting will be shown on the LCD, e.g. 0.0 kg.
3. Press the push-button again to toggle to lb.
4. Zero weight and the new weighing unit setting will be shown on the LCD, e.g. 0.0 lb

RS232 Output Port

1. RS232 output is always on after power up.
2. The default setting of the baud rate is 9600, 8, N, 1.
3. Once the weight is locked, the weight value is transmitted out the RS232 port.
4. Requires an optionally purchased interface cable to communicate to a printer or computer.

NOTE: Zero weight or Error messages are not transmitted out port RS232 port.

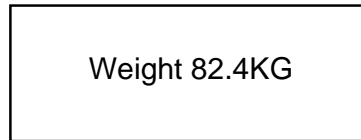Weight 82.4KG

Fig. 3: RS232 Output Format

Low Battery

When the battery voltage has dropped too low for accurate weighing, the *bAtt* message will be displayed for 2 seconds and the scale will automatically turn off. Remove the old batteries and replace with new batteries before continuing operation.

Error and Status Messages

NOTE: The error messages are shown for about 2 seconds and then the scale will turn off.

Display	Meaning
<i>Err0</i>	Incorrect initialization – The scale must be reset..
<i>Err1</i>	Unstable weight.
<i>Err2</i>	Overload – Scale weight exceeds scale capacity.
<i>bAtt</i>	Battery Low

Proper Disposal

When this device reaches the end of its useful life, it must be properly disposed of. It must not be disposed of as unsorted municipal waste. Within the European Union, this device should be returned to the distributor from where it was purchased for proper disposal. This is in accordance with EU Directive 2002/96/EC. Within North America, the device should be disposed of in accordance with the local laws regarding the disposal of waste electrical and electronic equipment.

It is everyone's responsibility to help maintain the environment and to reduce the effects of hazardous substances contained in electrical and electronic equipment on human health. Please do your part by making certain that the device is properly disposed of. The symbol shown to the right indicates that this device must not be disposed of in unsorted municipal waste programs.

FCC Compliance Statement

WARNING! This equipment generates, uses and can radiate radio frequency and if not installed and used in accordance with the instruction manual, may cause interference to radio communications. It has been tested and found to comply with the limits for a Class A computing device pursuant to Subpart J of Part 15 of FCC rules, which are designed to provide reasonable protection against such interference when operated in a commercial environment. Operation of this equipment in a residential area may cause interference in which case the user will be responsible to take whatever measures necessary to correct the interference.

You may find the booklet "How to Identify and Resolve Radio TV Interference Problems" prepared by the Federal Communications Commission helpful. It is available from the U.S. Government Printing Office, Washington, D.C. 20402. Stock No. 001-000-00315-4.

All rights reserved. Reproduction or use, without expressed written permission, of editorial or pictorial content, in any manner, is prohibited. No patent liability is assumed with respect to the use of the information contained herein. While every precaution has been taken in the preparation of this manual, the Seller assumes no responsibility for errors or omissions. Neither is any liability assumed for damages resulting from use of the information contained herein. All instructions and diagrams have been checked for accuracy and ease of application; however, success and safety in working with tools depend to a great extent upon the individual accuracy, skill and caution. For this reason the Seller is not able to guarantee the result of any procedure contained herein. Nor can they assume responsibility for any damage to property or injury to persons occasioned from the procedures. Persons engaging the procedures do so entirely at their own risk.

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