



## PZ30W/PZ60W Calibration Procedure

1. Power on display and base for normal weighing.
2. Press and hold **UNITS** key until *CR* is shown on display.
3. Display will then alternate between *LoAd* and a calibration weight.

PZ30W

*LoAd* and *20.00* lb    or    *LoAd* and *10.00* kg

PZ60W

*LoAd* and *50.00* lb    or    *LoAd* and *20.00* kg

To switch between LB and KG press **UNITS** key once.

4. Load scale to displayed weight.
5. Press TARE key once or Pass hand in front of Touchless Tare sensor.
6. Display will change to show *CR*.
7. Once calibration is complete, scale will return to normal weigh mode.
8. Remove weight from scale.
9. Scale is now calibrated and ready for normal use.

DETECTO  
102 E. Daugherty, Webb City, MO 64870 USA  
Ph: 417-673-4631 or 1-800-641-2008 • Fax: 417-673-2153  
[www.Detecto.com](http://www.Detecto.com)

Technical Support: 1-866-254-8261 • E-mail: [tech@cardet.com](mailto:tech@cardet.com)